

**CHAMELEON OSTOMY**

**Pro-Series Tube Cover Instruction Sheet**

**April 10, 2024**

**Included Parts:**

ID	Part	Qty
1	Chameleon Ostomy Pro Series Tube Cover (Available in Black or Gray)	1



**1. Pro Series Full Length Zipper Tube Cover**

Designed to conceal and protect drain tubes.

Our Tube Covers are designed to be long. This ensures the tube covers will fully conceal all currently available drain tubes. Excess cover is simply stowed inside your drain bag cover.

Orient the tube cover so that the tube cover logo is at the end closest to the drain bag. The starting point of the zipper is at the top end of the tube cover, closest to the urostomy pouch or where catheter exits the body.



Close the zipper by pulling towards the drain bag. The logo on the drain tube cover should be on the end closest to the drain bag.



If the user must remain connected to the urostomy pouch or catheter, the full-length separating zipper allows the tube cover to be installed and removed without having to disconnect. If the user can disconnect, the tube cover can remain zipped, and the tube can slide through the tube cover openings.

**Pro Tip:** You can coil excess tubing (and cover) inside the Privy XL so that you do not have excess tubing sticking out of the bag. Essentially, you control how much tubing you need exposed and coil excess inside the bag.

**Pro Tip:** You can use Quick Disconnect Fittings to add and remove tubing lengths as desired. For example, if you want to lengthen the tubing so that the drain bag is at foot of bed, making it easier to sleep on the user's side.



**Care Instructions:** Machine or Hand Wash, No Bleach, No Fabric Softeners. Clean localized dirt/soil with a water and a soft cloth. Air or machine dry low heat.

Questions? Contact Michael @ Chameleon Ostomy ([Info@ChameleonOstomy.com](mailto:Info@ChameleonOstomy.com))