

CHAMELEON OSTOMY

Privy Versa Hanging Strap Instruction Sheet

March 7, 2024

Included Parts:

ID	Part	Qty
1	Original Privy Bag	1
2	Versa Hanging Straps	2
3	Padded Shoulder Strap (Not Shown)	1




The Versa Straps:

The included Versa (Versatile) Straps can look a little overwhelming at first. They are designed to allow the user to hang the drain bag holder from a wide variety of objects. This includes mobility aids, bed rails, bed frames, desks, chairs, cushions, bicycle frames, etc.

Overview:

Each Versa Strap has an upper “loop section” and a lower “hook” section. The two sections are joined by a high strength quick release adjustable buckle. See below.

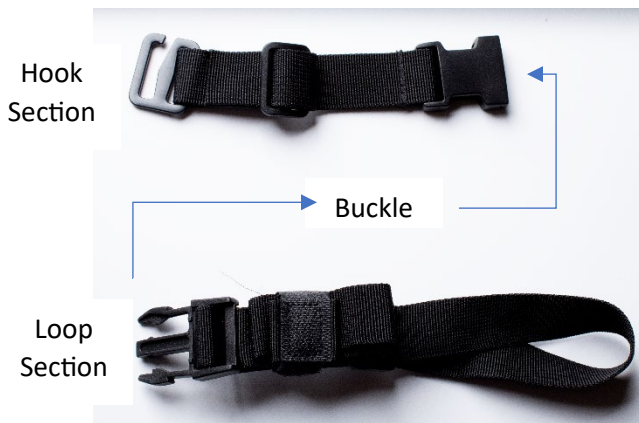

The Hook Section is has a slider to adjust the length of this section of the strap. Giving the user maximum adjustability and length. The G-Hook on the left is what attaches to the webbing hook on the drain bag holder.

The Loop Section is also adjustable. This section is what connects the strap to the object the drainage bag holder is being hung from.

Adjusting the length of the straps:

Feed the webbing through the sliders. It can look complicated until you become familiar with how the straps work. Loosen the webbing in the slider section by pushing the lower end of the webbing into or towards the slider. Once there is a little slack, you can pull the webbing through. Observe that one end of the strap is secured to the slider and then the remainder of the strap wraps through. Shorten or lengthen the strap by feeding webbing up or down through the slider.

Connecting the Strap to an object (Railing or Crossbar)

Around the object, buckle through the loops, and down.



It is recommended to adjust the strap length first. Then, attach the straps to the object. Last, hang the bag from the G-Hooks. This allows maximum room when hanging the strap in tight spaces. Note: the straps can be shortened by cutting with a sharp pair of scissors and then sealing the ends with a lighter. Be careful! This will prevent you from having to stow the excess webbing when the strap is in shorter configurations.

Connecting the bag to the Versa Strap: See Next Page

Connecting the bag to the Versa Straps:

The military style G-Hook makes it easy to secure the bag. When you take the weight off the bag by lifting the bag up, the G-Hook is easy to remove.

As seen below simply slide the open end of the G-Hook into the webbing loops at the top right and left sides of the drain bag. It does not matter if the hook faces left or right.

Note: Two webbing attach points are provided. The inside webbing is great for when the strap is supported from directly above. The Canted/Angled webbing can be used when the straps are connected to an object not directly over the bag, and off to the side.



The versa straps are versatile. You can adjust the lengths of the top and bottom sections independently. The straps can be adjusted separately, allowing the bag to be hung from supports that are not at the same level, while keeping the bag level if desired.

Our new Privy XL Features straps that are easier to use, with shorter strap lengths. We can also make custom straps if needed for your specific application.

Thank you,

Michael @ Chameleon Ostomy (Info@ChameleonOstomy.com)