

CHAMELEON OSTOMY

Privy XL Hanging Strap Instruction Sheet

April 9, 2024

Included Parts:

ID	Part	Qty
1	Privy XL Bag	1
2	Pro Series Color Matched Tube Cover	1
3	Padded Shoulder Strap	1
4	Hanging Straps	2
5	Heavy Duty Vinyl Bag Liner (Not Shown – It is in main compartment)	1



1. The Privy XL Bag:

When you first receive the bag, open the packaging, and look inside the front and main compartment and identify the components listed above.

Zippers: YKK Marine Grade, Auto-Locking Zippers. Lift the zipper pulls so that the pull is perpendicular to the zipper tape. Open and close each zipper. They should feel firm and operate smoothly. If desired a small amount of zipper lubricant may be applied. This is optional.

2. Pro Series Full Length Zipper Tube Cover

Our Tube Covers are designed to be long. Orient the tube cover so that the tube cover logo is at the end closest to the drain bag. The starting point of the zipper is at the top end of the tube cover, closest to the urostomy pouch or where catheter exits the body. If there is excess tube cover length, this extra length will remain unzipped and stowed/concealed inside the Privy XL during use. It is also possible to shorten the tube cover. It is recommended to tuck and sew the end of the zipper into the new seam, as this will prevent the zipper pull from coming off the zipper tape.



If the user must remain connected to the urostomy pouch or catheter, the full-length separating zipper allows the tube cover to be installed and removed without having to disconnect. If the user can disconnect, the tube cover can remain zipped, and the tubing can slide through the tube cover openings.

Pro Tip: You can coil excess tubing (and cover) inside the Privy XL so that you do not have excess tubing sticking out of the bag. Essentially, you control how much tubing you need exposed and coil excess inside the bag.

Pro Tip: You can use Quick Disconnect Fittings to add and remove tubing lengths as desired. For example, if you want to lengthen the tubing so that the drain bag is at foot of bed, making it easier to sleep on the user's side.

3. Padded Extra Long Shoulder Strap

Our padded shoulder strap is EXTRA long. Adjust the length of shoulder strap by feeding the webbing through the buckles. You may have to move one buckle through the padded shoulder strap, this is normal.

4. Hanging Straps

The included straps are designed to allow the user to hang the drain bag holder from a wide variety of objects. This includes mobility aids, bed rails, bed frames, desks, chairs, cushions, bicycle frames, etc.

Each Strap has one cam lock buckle and one mil spec style G-Hook.



Adjusting the length of the straps:

Release the cam lock buckle and slide the webbing through. Adjust to desired length and then fully close the cam lock buckle. It should close without much resistance and stay closed. When closed it securely holds the webbing in place, even with the drain bag fully loaded.

Connecting the Strap to an object (Railing or Crossbar)

Pro Tip: G-Hooks allow us to remove / install the Privy XL without having to disconnect the straps from the object they are connected to at the top. I have found it easier to connect the straps to the object and get them adjusted to the desired length and then attach the G-Hooks to the bag. This removes any weight and in tight quarters, such as when hooking up to a wheelchair frame, makes it much easier.

Release the cam lock lever. Release the webbing and wrap the webbing around the object. Slide the webbing back through the cam lock buckle. Then, close the cam lock lever completely.

Connecting the bag to the Versa Straps:

The military style G-Hook makes it easy to secure the bag. When you take the weight off the bag by lifting the bag up, the G-Hook is easy to remove.

CHAMELEON OSTOMY PRODUCT INFORMATION: PRIVY XL

As seen below simply slide the open end of the G-Hook into the webbing loops at the top right and left sides of the drain bag. It does not matter if the hook faces left or right.

Note: Two webbing attach points are provided. The inside webbing is great for when the strap is supported from directly above. The Canted/Angled webbing can be used when the straps are connected to an object not directly over the bag, and off to the side.



We can also make custom straps if needed for your specific application.

Thank you,

Michael @ Chameleon Ostomy (Info@ChameleonOstomy.com)